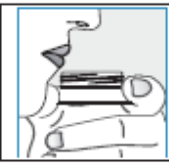


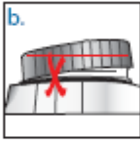
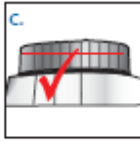



Saliva Sample Collection Date: _____ (dd/mm/yy) Time: _____ (hh:mm)

ORAGENE•DNA SELF-COLLECTION KIT USER INSTRUCTIONS

PLEASE DO NOT EAT, DRINK, SMOKE OR CHEW GUM FOR 30 MINUTES BEFORE GIVING YOUR SALIVA SAMPLE. DO NOT REMOVE THE PLASTIC FILM FROM THE CAP.

1		<p>Start giving your saliva sample.</p>	<p>To make more saliva, relax and rub your cheeks gently for 30 seconds. If you find it hard to create saliva, place ¼ tsp of white table sugar on your tongue.</p>
2	 <p style="font-size: 8px; text-align: center;">SIDE VIEW OF VIAL</p>	<p>Spit until the amount of liquid saliva (not bubbles) reaches at least the level shown in picture #2.</p>	<p>On average it takes approximately 5 to 10 minutes to provide a sample.</p>
3	<p>a.</p> 	<p>Put the container on a flat surface. Screw the cap onto the container. Make sure that the cap is closed tightly, as shown in picture #3c. If you have trouble closing the cap all the way, turn the cap slightly counter-clockwise then try to close it again.</p>	<p>b.</p>  <p>c.</p> 
4		<p>Mix gently for at least 10 seconds.</p> <p>Recycle outer packaging.</p>	

DIRECTIONS FOR MAILING THE SAMPLE

1. Record saliva collection date and time on the top of this sheet.
2. Place the closed collection vial (containing saliva) in the plastic bag. Remove as much air as possible and seal the bag.
3. Place the information sheet and the sealed plastic bag with the vial into the pre-stamped bubbled envelope. Seal the bubble envelope by removing the white strip.
4. Drop the envelope off into any Canada Post mailbox. There is no need to go to the post office.